

INSTRUCTION MANUAL

CA-A307E

Lifting pole

7C02126400A4

Preface

Thank you for purchasing our lifting pole.

This instruction manual describes precautions and how to safely use and operate the **lifting pole**.

- **To ensure safe and proper use of the lifting pole, be sure to read this instruction manual and the instruction manual for the suitable bed before use.**
- The safe and correct operating procedures should be explained not only to the person using the bed with **lifting pole** for medical treatment but also to the user's caregiver.
- After reading this instruction manual, store it in a place that can be easily accessed for later reference. Users and/or patients are requested to report information about any serious incident that occurs within the EU to the regulating authority of the member country of residence. Serious incident means any incident that directly or indirectly resulted in, may have resulted in, or may result in any of the following.
 - Death of a patient, user, or other person
 - Serious temporary or permanent adverse effects on the health of a patient, user, or other person
 - Serious public health threat
- Some of the descriptions in this instruction manual may differ from the product you have purchased due to model improvements.
- Values described for the operation range, dimensions, angles, and weights are subject to normal manufacturing tolerances. Except for special cases, words such as approximately and about are omitted in this instruction manual to improve readability.
- For any queries, contact Paramount Bed (see p. 5) or your distributor.


Intended use

- This lifting pole is designed to be attached to the bed for support to raise the body from the bed by attaching to the bed or for training too.
- Intended users : Medical professionals, patients
- Patient target group: Refer to the instruction manual of the medical device to be used together with this product.



Part names and part confirmation

- After unpacking, check that there are no missing or damaged parts. If any parts are missing or damaged, contact Paramount Bed or your distributor.

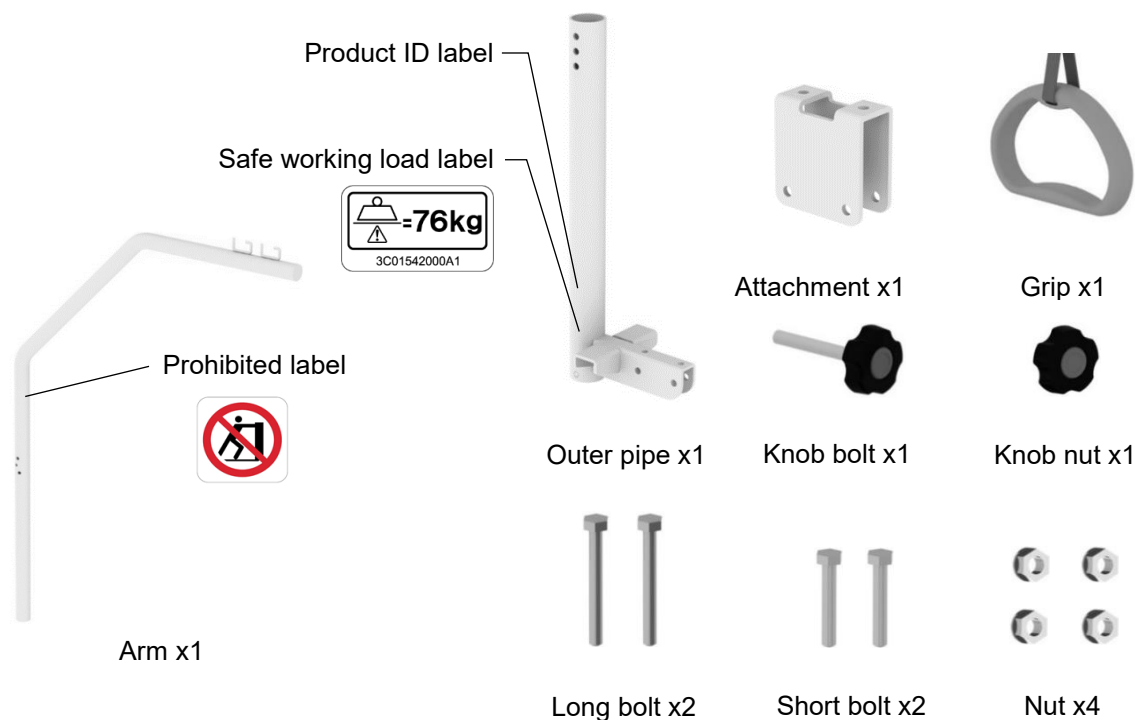

Specifications

<Specifications>

Subject to normal manufacturing tolerance

Product name	Lifting pole
Model No.	CA-A307E
External dimensions (mm)	1205 x 866 x 130 (when assembled)
Product weight (kg)	10
Materials	Made of steel, powder coating (part is resin mold item)
Load capacity (kg)	76

Environmental conditions	Range	
	When transported or stored	When in use
Temperature	-10°C to 50°C	5°C to 40°C
Humidity	20%RH to 90%RH at 30°C	20%RH to 90%RH at 30°C
Air pressure	700hPa to 1060hPa	700hPa to 1060hPa

<Classification>

Classification as medical device in Europe: Class 1

<Applicable standards>

Since this device is an accessory, for safety standards applied on this device, refer to the instruction manual of the suitable medical devices.

<Product lifetime>




The lifetime of this equipment is typically eight (8) years.

Lifetime is defined as the period during which the product will maintain the specified performance and safety, provided it has been maintained and operated in conditions of normal use in accordance with the requirements in this instruction manual.

Attaching the lifting pole

WARNING



- Load capacity is 76kg. Do not hang an article exceeding the load capacity because it may damage the lifting pole, resulting in an injury.
- The lifting pole can be used for support to raise the body from the bed by attaching to the bed. It can be used for training too.

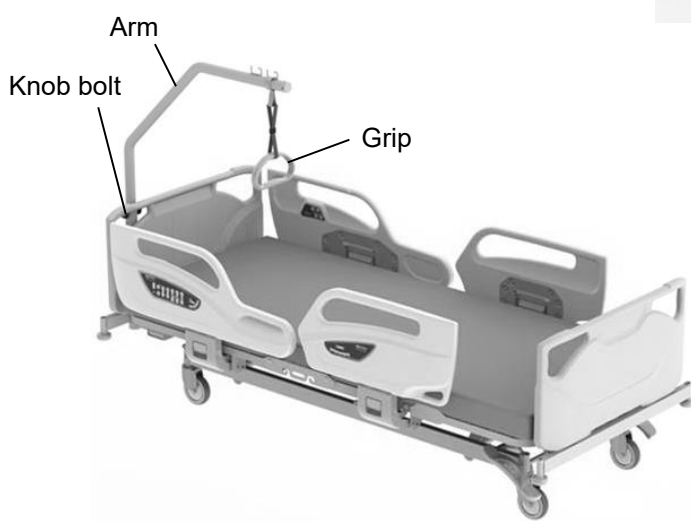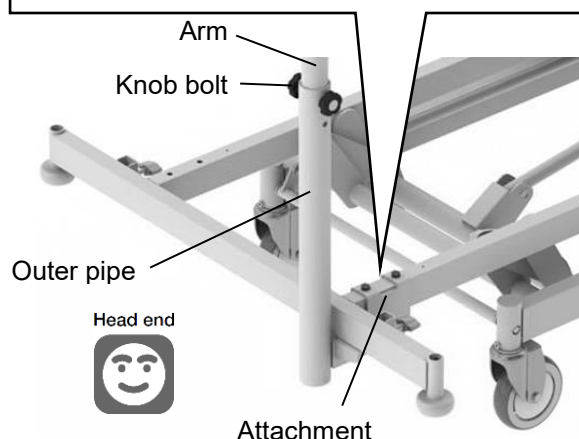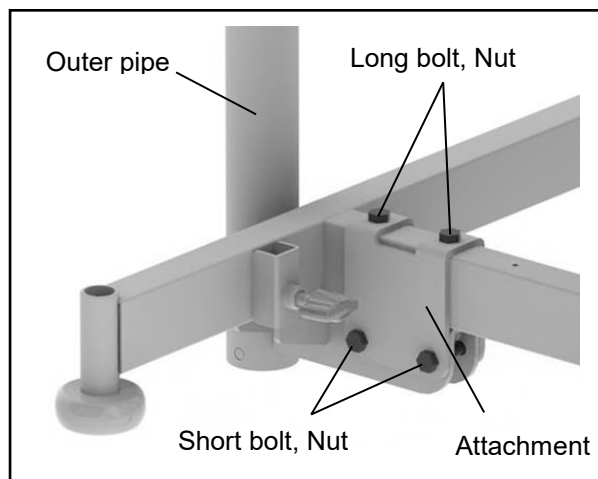

- ① Dethatch the headboard and raise the back section base.
- ② Set the attachment on the main frame of the bed.
- ③ Attach the outer pipe to the main frame and the attachment with 2 long bolts and 2 nuts as shown in figure.
- ④ Fix the outer pipe and the attachment with 2 short bolts and 2 nuts as shown in figure.
- ⑤ Attach the headboard and lower the back section base.
- ⑥ Inset the arm into the outer pipe and attach the grip to the arm.
- ⑦ Tighten the knob bolt and knob nut to fasten the arm.
- ⑧ Pull down the grip to make sure that the lifting pole is attached firmly.



- The lifting pole can be attached on both the right side and the left side of the bed.
- The arm can be set in three positions.

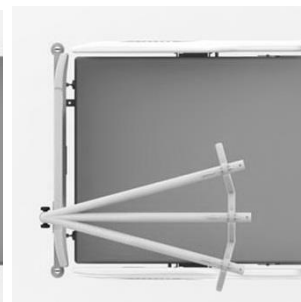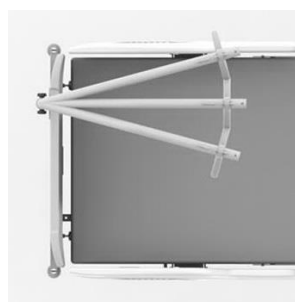

Daily maintenance/Long-term storage/Disposal

<Daily maintenance>

■ To disinfect parts.

Use one of the disinfectants listed below at the specific concentration, making sure to follow the instructions in the manuals or other materials available for the selected disinfectant.

76.9% - 81.4%	ethanol for disinfection
0.05% - 0.2%	benzalkonium chloride (e.g. Osvan)
0.05% - 0.2%	benzethonium chloride (e.g. Hiamine)
0.05%	chlorhexidine gluconate (e.g. Hibitane)
0.02% - 0.05%	sodium hypochlorite (e.g. Milton)

CAUTION

- Do not use volatile cleaning fluids (such as paint thinner, benzene or gasoline) and cresol. Discoloration or deterioration could result.



- The product cannot be used with an ozone sterilizer or an autoclave sterilizer, etc.

<Long-term storage>

- Do not place location that tend to be exposed to high temperatures, high humidity, low temperatures, dry or dusty conditions. Store at a temperature between -10°C and 50°C, humidity between 20%RH and 90%RH at 30°C, and atmospheric pressure between 700hPa and 1060hPa.
- Keep the instruction manual insert in a safe place so that they do not become lost.

< Disposal >

Dispose of the product in accordance with local laws and regulations. Contact Paramount Bed or your distributor (see "Contact information" as shown below) for any queries.

Product symbol definitions

■ The symbols below are used for this product.

Symbol	Description	Symbol	Description
	Date of manufacture		Serial number
	Manufacturer for CE mark		Country of manufacture (Made in China)
	CE mark		Model number
	Authorized representative in the European community. * Either of the symbols is used.		Refer to the instruction manual
			
	Safe working load		Medical device (This product is an accessory for medical devices and such accessory is required to be labeled as "Medical Device" under medical device regulations in Europe.)
	Do not lean against the lifting pole		
	This is a UDI (Unique Device Identification) code required by Unique Device Identification System designed to adequately identify devices through distribution and use. The information is coded in 2D bar code (GS1 Data Matrix). * In some countries or regions, do not use this symbol.		

After-sales service

1. When requesting repairs

Contact Paramount Bed or your distributor (see "Contact information" as shown below) for repair.

Details necessary when requesting repair

- Product name, model number, safety label number
- Date of purchase
- Details of fault or problem (please provide as much detail as possible)
- Your name, organization, address, and phone number

2. Minimum stock availability period

Paramount Bed keeps replacement parts (necessary to maintain oxygen cylinder holder performance) in stock for at least 8 years after the discontinuation of production.

3. Questions regarding after-sales service

Contact Paramount Bed or your distributor (see "Contact information" as shown below).

Contact information

For any queries, contact to the address or telephone number listed below.

[Location in Japan]	[Location in China]
 PARAMOUNT BED CO., LTD. 14-5, 2-chome, Higashisuna, Koto-ku, Tokyo, 136-8670, Japan TEL: +81-(0)3-3648-2961 FAX: +81-(0)3-3648-5781	PARAMOUNT BED (CHINA) CO., LTD. Plot A-105, Wuxi National Hi&New Tech Industrial Development Zone, Wuxi, Jiangsu, 214028, China TEL: +86-510-85323205 FAX: +86-510-8532-3212



NOTES

