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KC-0585 Power strip holder


- Before using this product, be sure to read this instruction manual and the instruction manual for the IV stand to fully understand how to safely and properly use this product.
- In addition to the person who uses this product, nurses and caregivers should also read this instruction manual.
- After reading this instruction manual, store it in a place that can be easily accessed for later reference.
- Values described for the operating range, dimensions, angles, and weights are subject to normal manufacturing tolerances. Except for special cases, words such as approximately and about are omitted in this instruction manual to improve readability.
- Some of the descriptions in this instruction manual may differ from the product you have purchased due to model improvements.
- If you have any questions, please contact Paramount Bed or your distributor.

INTENDED USE

This power strip holder is designed to assist power line management when installing a pump.


PART NAMES, CHECKING PARTS, AND SPECIFICATIONS

*) The illustration shows this product with a power strip attached. Provide a power strip yourself.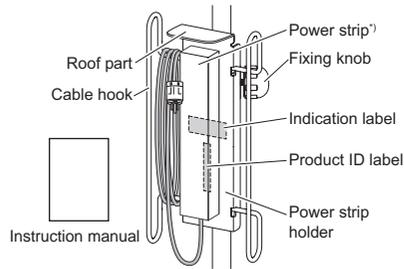

Model No. / Product Name	KC-0585 / Power strip holder
Dimensions (cm)	13 x 16 x 32.5
Product weight (kg)	1.1
Materials	Main unit: steel, fixing knob: ABS
Suitable products	IV stand (KC-50*/KC-51* series)

ATTACHMENT METHOD

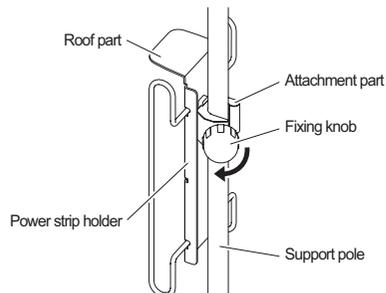Fit the attachment part of the power strip holder to the support pole of the IV stand, and then tighten the fixing knob properly to secure the power strip holder.

- When attaching the power strip holder, also refer to the instruction manual for the IV stand.
- Make sure that the fixing knob is tightened properly.
- Check the label affixed to the power strip holder and be sure to attach the power strip holder with the roof part at the top.

SAFETY PRECAUTIONS

WARNING

- Attach the power strip holder with the roof part at the top. If it is attached upside down by mistake and water or other liquid drips from a bag hanging from a hook of the IV stand, fire or electric shock may occur due to water getting on the power strip. This also may cause failure.
- Do not repair or modify the product by yourself. Doing so may cause an unexpected injury or failure.

CAUTION

- Do not hang heavy objects on the cable hook. The power strip holder may unintentionally become detached and fall, resulting in damage to the power strip or power strip holder.
- Be careful not to wind an excessive amount of cable. If you wind an excessive amount of cable, the cable may come off the cable hook and a foot or other body part may get caught on it, resulting in the IV stand falling over and becoming damaged or deformed or causing injury.
- Do not place any objects on the roof part. The objects may fall, causing damage.

PRODUCT SYMBOL DEFINITIONS

Symbol	Description
	Serial number
	Date of manufacture
	Manufacturer
	Refer to the instruction manual.
	Refer to the instruction manual.

Symbol	Description
	Be sure to carry out in accordance with the instructions.
	When using the power strip holder, check that it is attached so that the roof part is at the top.
	When using the power strip holder, make sure the roof part is not at the bottom.

DAILY MAINTENANCE AND LONG-TERM STORAGE

<Daily maintenance>

■ Removing stains

Perform steps ① to ③ below.

- ① Wipe off stains with a cloth soaked in a neutral detergent diluted with water and wrung out well.
- ② Wipe off any residual detergent with a cloth soaked in water and wrung out well.
- ③ Wipe off any residual moisture with a dry cloth.

■ Disinfecting parts

Use one of the disinfectants listed below at the specified concentrations by following the instructions in the instruction manual for the disinfectant.

- Up to 81.4% Ethanol for disinfection
- Up to 0.2% Benzalkonium chloride (e.g., Osvan)
- Up to 0.2% Benzethonium chloride (e.g., Hyamine)
- 0.05% Chlorhexidine gluconate (e.g., Hibitane)
- Up to 1% Sodium hypochlorite (e.g., Milton)

The following disinfectants can also be used.

- Super Sani-Cloth® Germicidal Disposable Wipe
- Clorox HealthCare® Bleach Wipe

* It is recommended to disinfect the power strip holder before another person uses the power strip holder.

Do not use an open sterilizer, autoclave, etc. to disinfect parts.

CAUTION

- Do not use volatile cleaning solutions (such as paint thinner, benzene, or gasoline) or cresol. Discoloration or deterioration may result.
- Do not use disinfectants other than those listed above. Damage or deterioration may result.
- Thoroughly wipe off parts with a damp cloth after using a neutral detergent or sodium hypochlorite. Not doing so may cause cracking, discoloration, or deterioration of the resin.

<Long-term storage>

- Avoid a location subject to high temperature or humidity, low temperature, dryness, or excessive dust. Store the power strip holder at a temperature between 14°F and 122°F (-10°C and 50°C) and humidity between 30% and 90% RH.
- Keep the instruction manual in a safe place so that it does not become lost.

AFTER-SALES SERVICE

For any queries about the after-sales service, contact your distributor or to the following address.

CONTACT INFORMATION

PARAMOUNT BED CO., LTD. Global Business Division
 14-5, 2-chome, Higashisuna, Koto-ku, Tokyo, 136-8670, Japan
 Tel: +81-(0)3-3648-2961/FAX: +81-(0)3-3648-5781
 For other contact information, please visit the website.
<https://www.paramount.co.jp/english/>