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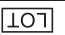
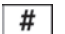
2021-10

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PRODUCT SYMBOL DEFINITIONS

The following symbols are used for this product.

Symbol	Description
	Serial number
	Manufacturer * It may be called "Legal Manufacturer" or "Brand Owner" etc.
	<ul style="list-style-type: none"> Country of manufacture (Made in Japan) * In the country of origin, the site acts as the manufacturer.
	Refer to the instruction manual.
	Be sure to carry out in accordance with the instructions.
	Unique Device Identifier This is a 2D Code (GS1 DataMatrix) that encodes the UDI (Unique Device Identifier) required by the Unique Device Identification System designed to adequately identify devices through distribution and use.

Symbol	Description
	Do not stand on the step or base.
	Adjust the height while squeezing the vertical stopper knobs.
	Turn the grip hold while pushing the angle adjustment button with a finger.
	Lot number
	Model number

AFTER-SALES SERVICE

1. When requesting repairs

Contact your distributor or directly contact Paramount Bed (see Contact Information below).

Details necessary when requesting repair

- Product name, model number, product ID label number (for the affixed location, see "PART NAMES" on the front cover)
- Date of purchase
- Details of the fault or problem (please provide as much detail as possible)
- Your name, organization, address, and phone number

Consumable parts

- Casters are consumable parts.

2. Minimum stock availability period

Paramount Bed keeps replacement parts (necessary to maintain performance of the product) in stock for at least 8 years after the discontinuation of production.

3. Questions regarding after-sales service

Contact your distributor or directly contact Paramount Bed (see Contact Information below).

Disposal

Dispose of the product according to the local rules and regulations.

Contact Information

For any queries, please feel free to contact your distributor or to the following addresses.

[Location in Japan (Manufacturer)]	[Location in Japan (Manufacturing premises)]
 PARAMOUNT BED CO., LTD. 14-5, 2-chome, Higashisuna, Koto-ku, Tokyo, 136-8670, Japan Tel: +81-(0)3-3648-2961 Fax: +81-(0)3-3648-5781	 PARAMOUNT BED CO., LTD. CHIBA FACTORY 2078, Shirahata, Sammu-shi, Chiba, 289-1306, Japan

For other contact information, please visit the website.
 Website: <https://www.paramount.co.jp/english/>

DAILY MAINTENANCE AND LONG-TERM STORAGE

>Daily maintenance

■ When removing stains from parts

Perform steps ① to ③ below.

- ① Wipe off stains with a cloth soaked in a neutral detergent diluted with water and wrung out well.
- ② Wipe off any residual detergent with a cloth soaked in water and wrung out well.
- ③ Wipe away any leftover moisture with a dry cloth.

■ When disinfecting parts

Use one of the disinfectants listed below at the specified concentrations by following the instructions in the instruction manual for the disinfectant.

Up to 81.4%	Ethanol for disinfection
Up to 0.2%	Benzalkonium chloride (e.g., Osvan)
Up to 0.2%	Benzethonium chloride (e.g., Hyamine)
Up to 0.05%	Chlorhexidine gluconate (e.g., Hibitane)
Up to 1.0%	Sodium hypochlorite (e.g., Milton)

The following disinfectants can also be used.

- Super Sani-Cloth® Germicidal Disposable Wipe
- Clorox HealthCare® Bleach Wipe

* It is recommended to disinfect the IV stand before another person uses the IV stand.

- Do not use volatile cleaning solutions (such as paint thinner, benzine, or gasoline) or cresol. Discoloration or deterioration may result.
- Do not use disinfectants other than those listed above. Damage or deterioration may result.
- Thoroughly wipe off parts with a damp cloth after using a neutral detergent or sodium hypochlorite. Not doing so may cause cracking, discoloration, or deterioration of the resin.

- Do not use an ozone sterilizer, autoclave, etc. to disinfect parts.

<Long-term storage


- Avoid a location subject to high temperature or humidity, low temperature, dryness or excessive dust. Store the IV stand at a temperature between 14°F and 122°F (-10°C and 50°C) and humidity between 30% to 90% RH.
- Keep the instruction manual in a safe place so that it does not become lost.

SPECIFICATIONS

Product name	IV stand
Model No.	KC-516U/KC-518U/KC-516/KC-518
Dimensions	Base part max diameter of Ø18.7 in. (47.4 cm) x total height of 52.0 in. (132.2 cm) to 78.6 in. (199.7 cm)
Product weight	KC-516U/KC-516: 11.7 lb (5.3 kg) KC-518U/KC-518: 9.3 lb (4.2 kg)
Material	Pole
	Stainless steel
Max load weight	Base, grip hold, vertical stopper, hanger top
	Hooks
	Sliding pipe (upper pole)
	Pole (upper + lower pole)
	27.6 lb (12.5 kg)


- Read the instruction manual for each suitable product.
 - Some of the descriptions in this instruction manual may differ from the actual product due to specification changes.
- For any queries, contact Paramount Bed or your distributor.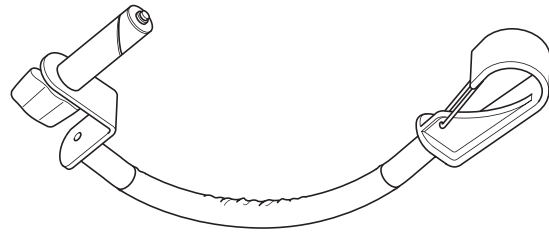


■ **Linking arm**
(KC-0582/KC-0583 for KC-516/KC-518
KC-0582U for KA-516U/KC-518U)

The IV stand can be linked with a compatible bed, stretcher, or wheelchair.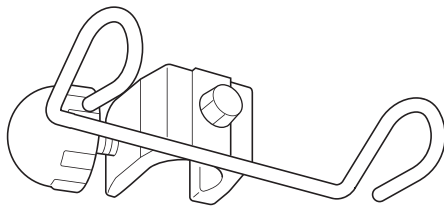


■ **Urine collection bag holder**
(KC-058A for KC-516/KC-518
KC-058AU for KC-516U/KC-518U)

A urine collection bag holder can be attached to the IV stand to hang a urine collection bag.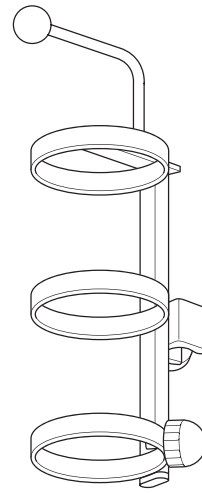

■ **Oxygen cylinder holder**
(KC-0581 for KC-516/KC-518
KC-0581U for KC-516U/KC-518U)

An oxygen cylinder holder can be attached to the IV stand to hold an oxygen cylinder.

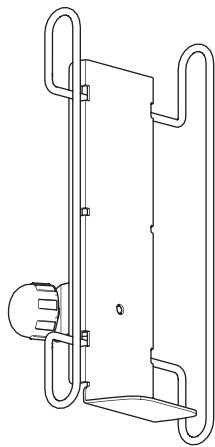

■ **Power strip holder (KC-0585)**

A power strip holder can be attached to the IV stand to attach a power strip.


● Check that the hanger top and hooks are not damaged or deformed before use.


■ Adjusting the grip hold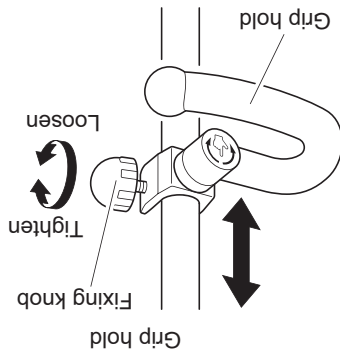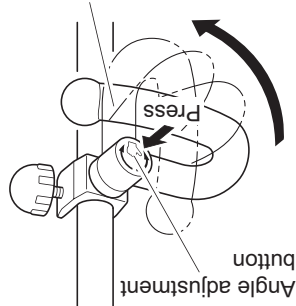

■ Adjusting the hanger top height

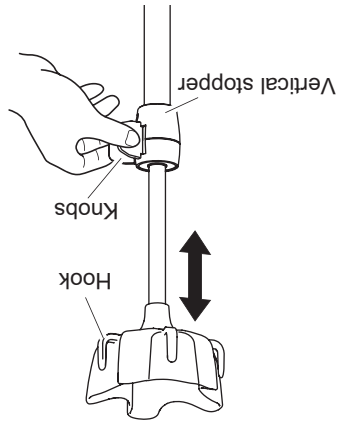■ Disassembling a caster
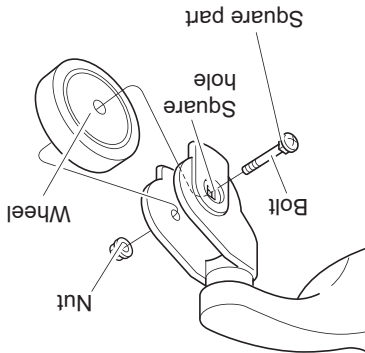
<Angle adjustment>
Turn the grip hold while pushing the angle adjustment button with a finger. The grip hold locks in place every 30°. There are 12 levels of adjustment (360°). Check that the grip hold is in the locked state before using the product.

<Height adjustment>
Loosen the fixing knob, adjust the grip hold to the desired height, and tighten the fixing knob properly so that the grip hold will not shift out of place.

Adjust the height while squeezing the vertical stopper knobs as shown in the illustration on the left.

● Check that there is nothing in the way around the hanger top before adjusting its height.

If a wheel is clogged with foreign matter such as dust and hair and does not rotate well, use the following procedure to disassemble the caster and remove the foreign matter.

① Loosen the nut that secures the caster and then pull out the bolt and remove the wheel.

② Remove the foreign matter and then reinsert bolt and tighten the nut properly. After that, check that there is no play in the caster.

● Be sure to insert the bolt from the square hole side and align the square part of the bolt with the square hole when inserting the bolt.

● The recommended tightening torque for assembly when using an electric tool is 39.8 lbf-in (4.5 N·m) to 57.5 lbf-in (6.5 N·m).


WARNING

Indicates that improper handling by ignoring this indication may cause death or serious injury (broken bones, pressure being applied, or paralysis).

- **Do not adjust the IV stand height while items are hanging on the hooks.**
 - The items may suddenly fall from the hooks, causing injury, or the IV stand may be damaged or deformed.
- **When attaching an infusion pump or other device, take care that the maximum load weight is not exceeded.**
 - The maximum load weight for the sliding pipe is 7.7 lb (3.5 kg). Furthermore, the maximum load weight for the pole (both the sliding pipe and support pole combined) is 27.6 lb (12.5 kg). If used with the maximum load weight exceeded, the IV stand may fall over and become damaged or deformed, causing injury.
- **When attaching an infusion pump or other device, check the stability of the IV stand.**
 - The IV stand may fall or the sliding pipe may suddenly lower, causing injury. When attaching a device to the sliding pipe, be sure to attach it by placing it on the vertical stopper.
- **When attaching a device (infusion pump, etc.), read the device manufacturer's instruction manual.** 
- **Before using the IV stand, check that the fixing knob of the grip hold is properly tightened.**
 - Before using the IV stand, be sure to check that the fixing knob of the grip hold is properly tightened. If the fixing knob is not tightened enough, the grip hold may fall, causing injury.
- **Do not use the IV stand as a support to stand up.**
 - The IV stand may fall over, causing injury.
- **Do not apply excessive force to the hanger top and hooks.**
 - The hanger top and hooks may be damaged or deformed, and an item may unexpectedly fall, causing injury.
- **Check that the hanger top and hooks are not damaged before use.**
 - If they are used while damaged, an item may fall, causing injury, etc.
- **Stop use if there is a failure or damage.**
 - If the IV stand fails or is damaged, immediately stop using it and contact your distributor or Paramount Bed (see p. 7) for repair. An injury may occur due to the IV stand being deformed or otherwise damaged.
- **Do not use the product near thermal appliances (heaters, etc.) or in a location exposed to direct sunlight.**
 - Doing so could cause the IV stand to be damaged or deformed, or could even cause it to catch fire.
- **Do not repair or modify the IV stand or its components by yourself.**
 - Doing so may cause an accidental injury or damage.
- **Do not hang items on the grip hold.**
 - If items are hung on the grip hold, the IV stand may lose its balance and fall, causing injury.
- **Use care when setting the angle of the grip hold.**
 - Depending on the angle of the grip hold, the hand of a person with a weak gripping strength may slip off the grip hold and the person may fall, causing injury. For such persons, use the IV stand with the grip hold positioned horizontally.
- **When moving the IV stand, do not move it quickly.**
 - The IV stand may fall over, causing injury.

CHECKING PARTS

■ After unpacking, check that there are no missing or damaged parts. If any parts are missing or damaged, contact Paramount Bed or your distributor.

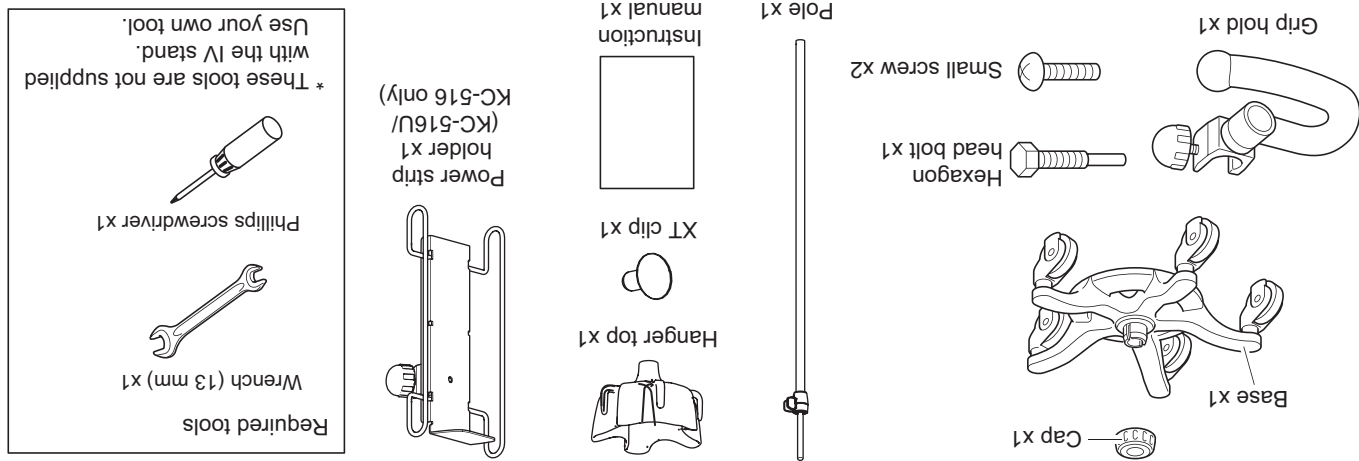SAFETY PRECAUTIONS

■ **Do not stand on the step or base.**
 ● The IV stand may fall over, causing injury, or it may be damaged or deformed.

■ **When stopping movement with a foot, do not remove your heel from the floor.**
 ● When stopping movement with a foot while performing a task such as changing the infusion bag, do not remove the heel of the foot that is depressing the step from the floor. The IV stand may fall over, causing injury.

■ **Pay close attention when moving the IV stand on a rough surface or over a bump or narrow gap, or when using it on a slope.**
 ● The IV stand may fall over, causing injury. When there are bumps, lift the IV stand to pass over them. Be especially careful when an infusion pump or other device is attached because the IV stand will fall over easily in such a state.

■ **When moving the IV stand, hold the grip hold.**
 ● If the IV stand is held in a location other than the grip hold, it may fall over, causing injury.

■ **Do not use the IV stand in a location that is not level.**

● The IV stand may move by itself and fall over or the item hanging on it may fall off, causing injury.

■ **Do not use the IV stand for purposes other than those for which it is intended.**

● If the IV stand is used for a purpose such as a walker or handrail, it may fall over, causing injury.

■ **Do not hang any items exceeding the maximum load weight on the hooks.**

● The maximum load weight per hook is 5.5 lb (2.5 kg) (total of 5.5 lb (2.5 kg) for all hooks).

When items that exceed the maximum load weight are hung, the hooks may break and fall down or the IV stand may fall, causing injury.

Load example: Two 1,000 ml infusion bags and one 500 ml infusion bag (total 5.5 lb (2.5 kg))

Preface

Thank you for purchasing our IV stand. This instruction manual describes precautions and how to safely use the IV stand.

- Before using the IV stand, be sure to read this instruction manual to fully understand how to safely and properly use the IV stand.
- In addition to the person who uses the IV stand, nurses and caregivers should also read this instruction manual.
- After reading this instruction manual, store it in a place that can be easily accessed for later reference.
- Some of the descriptions in this instruction manual may differ from the product you have purchased due to model improvements.
- Values described for the operating range, dimensions, angles, and weights are subject to normal manufacturing tolerances. Except for special cases, words such as approximately and about are omitted in this instruction manual to improve readability.
- If you have any questions, please contact Paramount Bed or your distributor.

INTENDED USE

■ The IV stand is designed to be used to attach an infusion bag when administering an IV drip.

PART NAMES